


SD BIOLINE Dengue NS1 Ag and IgG/IgM Test

[Dengue NS1 Ag]

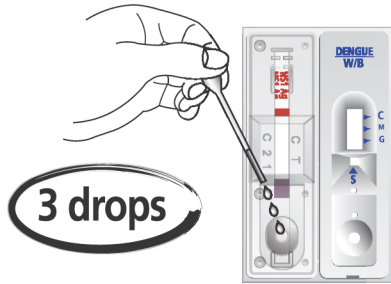


Using disposable dropper provided, add 3 drops (about 100 μ l) of serum, plasma or whole blood into the sample well marked "S".


Interpret test results in 15~20 minutes.




[Dengue IgG / IgM]

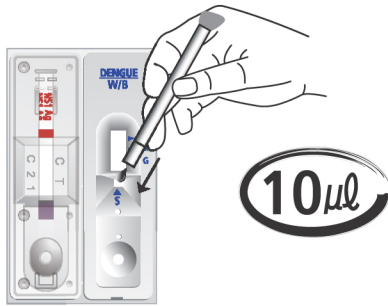

Using capillary pipette provided, add 10 μ l of serum, plasma or whole blood into the sample well marked "S".

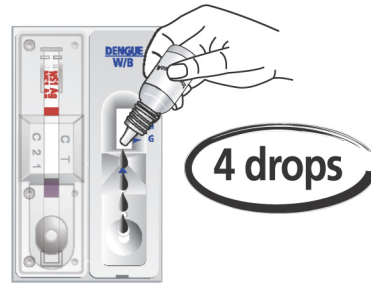

Put 4 drops of assay diluent into the assay diluent well round shaped.

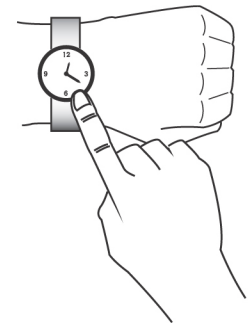

Interpretation

[Dengue NS1 Ag]

Negative
The presence of only one color line within the result window indicates a negative result.

Positive
The presence of two color lines ("T" band and "C" line) within the result window in left-side, no matter which line appears first, indicates a positive result.

Invalid
-No control (C) line in result window.
-It is recommended that the specimen be re-tested.

[Dengue IgG / IgM]

Positive

1. IgM positive
(Primary dengue infection)
-Two lines "C" and "M" in result window.
-It is positive even if "M" line is weak.

2. IgG positive
(Secondary or past dengue infection)
-Two lines "C" and "G" in result window.
-It is positive even if "G" line is weak.

3. IgG and IgM positive
(Late primary or early secondary dengue infection)
-Three lines "C", "M" and "G" in result window.

Negative
(No dengue infection)
-One line "C" in result window at right

Invalid
-No control (C) line in result window.
-It is recommended that the specimen be re-tested.