

OraQuick® the rapid oral fluid test helps



Diagnose the Undiagnosed

OraQuick® HCV

Rapid HCV testing with
>99% accuracy*

**The first and only rapid
HCV antibody test for**

- Fingertick Whole Blood
- Venous Whole Blood
- Plasma or Serum
- Oral Fluid



OraQuick® Rapid
Antibody
Test
HCV

*Whole blood, plasma, and serum.

OraQuick® the rapid oral fluid test helps



Diagnose the Undiagnosed

Simple. Accurate. Versatile.

- OraQuick®'s easy-to-use, three-step platform gives you rapid results in 20 minutes
- Clinical performance with >99% accuracy* and reactivity with all HCV genotypes
- Versatile sample collection enables testing in both clinical and nonclinical settings

Rapid Oral Fluid Antibody Testing Results in 20 Minutes

Step 1

Oral Fluid sample. Swab the lower and upper gum **once**.



Step 2

Insert device into vial containing buffer solution.



Step 3

Read results between 20 and 40 minutes.



Non-Reactive



Reactive

OraQuick® HCV Accurate Clinical Performance¹

| Sample | Sensitivity | Specificity |
|-------------------|----------------------------------|----------------------------------|
| Fingerstick Blood | 99,7% (95%CI: 99,0-100,0) | 99,9% (95%CI: 99,6-100,0) |
| Venous Blood | 99,7% (95%CI: 99,0-100,0) | 99,9% (95%CI: 99,5-100,0) |
| Plasma | 99,9% (95%CI: 99,3-100,0) | 99,9% (95%CI: 99,5-100,0) |
| Serum | 99,9% (95%CI: 99,3-100,0) | 99,9% (95%CI: 99,6-100,0) |
| Oral Fluid | 98,1% (95%CI: 96,9-99,0) | 99,6% (95%CI: 99,2-99,9) |

For more product information, visit OraSure.com or OraQuick.com
To order, contact your local distributor.



Ordering Information

1001-0270 Box of 25 Tests
1001-0274 Box of 100 Tests

OraSure Technologies

220 East First Street
Bethlehem, PA 18015 USA
phone: 610.882.1820
web: www.OraSure.com

* Whole blood, plasma, and serum.

† 4 package inserts are included, each of which covers a different set of languages.

Reference: 1. OraQuick® HCV Rapid Antibody Test CE Package Insert. OraSure Technologies. Bethlehem, PA USA. Please refer to the package insert for complete information and instructions on the proper use of the OraQuick® HCV Rapid Antibody Test.

©2011 OraSure Technologies. U.S. patent #7,192,555 and various international and U.S. patents pending. OraQuick® is a registered trademark of OraSure Technologies Item# HCV0084 Rev. 02/12 Made in the USA



Qarad bvba
Cipalstraat 3
B-2400 GEEL
BELGIUM