
CODAN

Contrelle® Activgard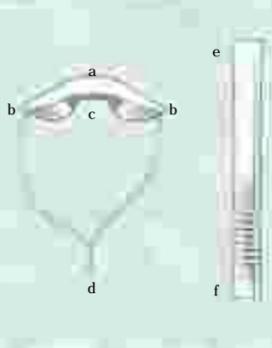

D

Contrelle® Activgard

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GB

Contrelle® Activgard is a new but well-documented appliance for women suffering from stress incontinence. That is, women who experience involuntary loss of urine when coughing, lifting and running.

Contrelle® Activgard

- Contrelle® Activgard
- Wings
- Depression in middle
- String
- Applicator
- Rings

Simple and effective help
Everything is difficult in the beginning. But if you relax and give yourself plenty of time, practice makes perfect. Contrelle® Activgard is inserted into the vagina, where it supports the neck of the bladder. It gives back the

voluntary control of the bladder that stress-incontinent women lose because of weakness in the muscles of the pelvic floor, allowing them to lead a more active life. Contrelle® Activgard can be used whilst running, swimming or taking other forms of exercise.

Contrelle® Activgard is available in three sizes. Try starting with the smallest and continue until you find the size that feels most comfortable. Remember, each Activgard can only be used once. It is inserted with an applicator, which makes the procedure clean and hygienic.

Instructions

1 Before you start, you should read the instructions carefully and have this leaflet to hand on the floor, a wash basin or some other convenient place so you can read it as you follow each step.
• Wash your hands before opening the package.
• Soak the Activgard thoroughly under running cold water. Remember to squeeze it 9 or 10 times to make sure it is properly saturated – then it will not absorb moisture from the vagina.
• When saturated, squeeze out any excess water.

2 Put the Activgard on the applicator as follows:

- Put the string around the bottom end of the applicator, with the knot in the middle of the small hollow.
- Pull the Activgard to the end of the applicator so its top rests in the depression in the middle of the inner surface of the Activgard.

3 • Adopt a comfortable position that makes it easy to insert the Activgard.

- Hold the applicator by the rings and turn the Activgard so that its wings point towards the urethra and anus – not towards the thighs.
- Part the labia with one hand and insert the Activgard until your fingers touch your body.
- Ease the string off the applicator

and withdraw it.
• Wash the applicator carefully, ready for future use.

removed before urination. To remove the Activgard, pull gently on the string. Do not flush it down the toilet.

- Important**
- It is important that you relax and give yourself plenty of time when you insert the Activgard. If you are tense, the muscles in your vagina will be tense too.
 - If you experience bleeding, irritation of any discomfort when using the Activgard, contact your doctor.
 - A little penetration cream (available at any chemist's) can make insertion easier.
 - The Activgard should be changed every 4 to 6 hours during menstruation.
 - The menopausal women, the use of hormone cream in the vagina can

Contrelle® Activgard can be used for up to 16 hours a day and need not be

- make it easier to use the Activgard.
- Do not use the Activgard if you are suffering from any infection in the vagina.
- Consult your doctor about use during pregnancy.
- The Activgard should be removed prior to intercourse.
- Do not use anything other than the applicator to insert the Activgard.
- Consult your doctor if you feel no benefit from the Activgard or if you encounter difficulty in removing it.
- The Activgard should be used only once.
- CODAN accepts no responsibility for any damage or injury caused by Contrelle® Activgard and applicator when used in any way other than that described in the accompanying instructions.

Toxic shock syndrome, TSS, is a very rare condition caused by the bacterium, Staphylococcus aureus. About half the cases registered occur among menstruating women who use tampons. Most cases are found among younger women who use highly absorbent tampons. When Contrelle® Activgard has been fully saturated with water before use, it absorbs no moisture from the vagina. Although the risk of TSS is very small, it is important to be able to recognise the symptoms as the condition is serious and should receive immediate treatment. Typical TSS symptoms are a high temperature (usually exceeding 39°C, 102°F), a sunburn-like rash, diarrhoea, vomiting, a sore throat and dizziness. If you should contract symptoms such as these, remove the Activgard and contact your doctor

immediately. Inform your doctor that you have been using the Activgard.

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