Intended Use

Clearview EASY HCG is intended for the qualitative detection of human chorionic gonadotrophin (hCG) in urine as an aid in the diagnosis of pregnancy. For in vitro diagnostic use only.

Summary

HCG is a glycoprotein hormone produced by the blastocyst^{1,2}. HCG normally begins to be detected in the urine from 7 days after conception. The sudden rapid rise in concentration of hCG following conception makes it an excellent marker for pregnancy^{3,4}.

Principle of Test Procedure

Urine is added to a sample wick, which contains blue beads attached to an anti-hCG monoclonal antibody. Urine mobilises the labelled antibodies and moves up the test strip, which contains regions of immobilised antibody. A blue Control Line appears in Region R (see Figure 1) to show the test has worked correctly. If hCG is present at 25mIU/ml or greater, a blue line should become visible in the test Region T. No line in Region T indicates the test is negative.

Kit Contents and Storage

20 **Clearview EASY HCG** tests each containing murine monoclonal antibodies to hCG and goat antibodies. Store at 2-30°C. Do not use after the expiry date.

Sample Collection and Storage

A urine sample collected any time of day is suitable but a first morning urine sample is recommended. Urine samples must be collected in **clean**, dry plastic or glass containers. Store samples in the refrigerator (2-8°C) or frozen. Bring to 18-30°C before testing. No sample preparation is required, although particulate matter should be allowed to settle before testing. **Clearview EASY HCG** has not been validated for use with samples containing preservatives.

Assay Procedure

Ensure all **Clearview EASY HCG** tests and samples are at 18°C to 30°C. When ready to test, tear open a foil wrapper and remove the test stick. Do not use test sticks that have become wet or have a damaged foil pouch. Follow **one** of the following procedures:

METHOD A – DIP

- Dip the sample wick (B) into the sample (D) to point C, as indicated in the diagram.
- Hold in place for 15 seconds.
- Remove the test stick from the sample.
- Replace the cap (A), avoiding inverting the test stick.
- Read result within 3 to 10 minutes.

METHOD B – DIP AND LEAVE

- Dip the sample wick (B) into the sample (C) until half the wick is immersed, as shown in the diagram.
- Hold in place for 3 to 10 minutes.
- Remove the test stick from the sample.
- Replace the cap (A), avoiding inverting the test stick.
- Read result straight away.

METHOD C – PIPETTE

- Place the test on a clean, dry, flat surface with the absorbent side of the sample wick (B) facing upwards.
- Pipette 100µl of the urine sample onto the wick.
- Remove the cap (A) and place over the wick, avoiding inverting the test stick.
- Read result within 3 to 10 minutes.

METHOD D – URINE STREAM

- Ask the patient to hold the sample wick (B) pointing downwards in their urine stream (E) for 3 to 7 seconds **only**.
- After 3 to 7 seconds the patient should remove the sample wick from the urine stream and place the cap (A) over the wick without inverting the test stick.
- Place the test stick on a flat surface.
- Read result between 3 and 10 minutes.

Interpretation of Results


The test must be read 3-10 minutes after beginning the test, whichever test procedure is used. See Figure 1.

Figure 1

Positive Result

Negative Result

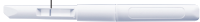- A blue Control Line (R) **MUST** be present at 3 minutes otherwise the test must be repeated with a new test stick.
- A blue Control Line in Region R and a blue Test Line in Region T indicates a **positive result (+)**. A difference in the intensity of the lines has no effect on the interpretation.
- One blue Control Line in Region R indicates a **negative result (-)**. Dispose of the test stick safely after use.

Limitations of the Test

1. Occasionally, samples containing <25mIU/ml hCG may test positive but samples containing <5mIU/ml should be negative. **Clearview EASY HCG** has been shown to be over 99% accurate.
2. A negative result may be obtained if a urine sample is too dilute. If pregnancy is still suspected, it is recommended the patient should be retested 48-72 hours later using an early morning sample.
3. Positive results from very early pregnancy may later prove negative as natural termination is estimated to occur in 31% of all conceptions⁵. It is therefore recommended that weak positive results are re-tested 48-72 hours later with early morning urine.
4. Concentrations of hCG are generally lower in ectopic pregnancy than expected normal values for a given gestational age. Abnormal pregnancy cannot be distinguished from normal pregnancy by hCG levels alone⁶.
5. HCG remains elevated for a time after pregnancy⁷. Pregnancy tests carried out less than 3 weeks after giving birth or 9 weeks after natural loss or termination may need further evaluation.
6. A number of conditions other than pregnancy can cause elevated levels of urinary hCG e.g. menopause⁸, trophoblastic disease, and certain non-trophoblastic neoplasms⁹.
7. Drugs containing hCG (for example some fertility drugs) may interfere with **Clearview EASY HCG**, and produce misleading results.
8. False positive pregnancy tests have been observed in patients with enterocystoplasties⁵. It is recommended that all positive pregnancy test results from such patients are confirmed with a blood test.
9. Do not open the foil pouch until ready to test.

Expected Values

Urine samples from healthy males and non-pregnant females generally contain less than 10mIU/ml hCG¹⁰. On the first day of the first missed period, the levels of maternal urinary hCG are normally 50-250mIU/ml. At the end of the first trimester hCG levels peak at up to 200,000mIU/ml¹¹. **Clearview EASY HCG** has been shown to produce positive results with samples containing up to and including 250,000 mIU/ml hCG, which is higher than the maximum level expected during a typical pregnancy.

Performance Characteristics

Sensitivity

Clearview EASY HCG can detect hCG in a urine sample at concentrations of ≥25mIU/ml. This sensitivity has been determined against the 4th International hCG Standard (WHO). Samples containing <5mIU/ml hCG should be negative.

Specificity

A range of substances were added to both positive and negative urine samples at the following concentrations and no cross-reactivity was demonstrated:

● LH (500mIU/ml)	● Haemoglobin (3.6mg/ml)	● Caffeine (0.2mg/ml)
● FSH (200mIU/ml)	● Albumin (12mg/ml)	● Salicylic acid (0.2mg/ml)
● TSH (1000µIU/ml)	● Bilirubin (10µg/ml)	● Pregnenediol-3-glucuronide (1µg/ml)
	● Ascorbic acid (2mg/ml)	● Oestrone-3-glucuronide (1µg/ml)
	● Glucose (10mg/ml)	
	● Uric acid (0.1mg/ml)	

Accuracy

In a study conducted using 99 urine samples collected from women for the purpose of pregnancy testing, **Clearview EASY HCG** has been shown to be over 99% accurate, relative specificity and sensitivity were found to be 100%. The study results are summarised below:

[hCG] mIU/ml*	Clearview EASY HCG	
	+	-
< 5.0	0	71
5.0 – 24.9	5	0
≥ 25	23	0

*Assessed by quantitative enzyme immunoassay.

Advice Line

Further information can be obtained from www.clearview.com, your distributor, or UK customers can call Unipath Customer Services on **08705 134952**.

Unipath, Clearview and the Diamonds device are trademarks. Protected by Patents including EP291194, EP560411, AU626207, AU656966, AU656967 and Design Registrations GB2042109, GB2042110. Corresponding and other rights granted and pending.

Unipath, Clearview and the Diamonds device are trademarks.

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©Unipath 2003
Unipath, Clearview und das Rhomben-Logo sind Warenzeichen. Unipath Ltd, Bedford MK44 3UP, UK.